oxygen based oral science

blue^m oral gel instructies (NL)

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blue^m oral gel instructions for use (EN)

Description:

Once applied on the affected area, the gel will release oxygen creating a partially high oxygen pressure (pO₂), which aids in restoring the micro- environment for a healthy oral cavity. blue^m oral gel has a white and matte colour and the odour is sweet. blue^m oral gel is not intended for use in conjunction with other medical devices, medicines or other medical technologies. blue^m oral gel is not recommended to be used with metal subjects such as metal needles, metal implants, or metal prosthesis, with the exception of titanium and other noble metals.

Intended use:

The intended purpose of blue^m oral gel is to restore normal levels of oxygen in the oral cavity. blue^m oral gel is intended to be applied at: pockets, bleeding gums, or wounds, which can result from extractions, oral surgery and placement of implants, chemotherapy or false teeth.

Intended users:

The intended users are either Healthcare Professionals (HCP) working within dental healthcare or persons with dental issues such as pockets, bleeding gums and/or wounds that can result from extraction, oral surgery and placement of implants, chemotherapy or false teeth in their mouth.

Application:

First dry the affected area with a cotton

swab so that the gel will remain better on-site. Then use the tube to apply blue^m oral gel at the place where it is needed: maximally 1 ml of oral gel at each location, which is approximately the size of a pea. Do not apply more than 5 ml of oral gel per day. Leave the gel on the site as long as possible, but rinse your mouth approximately one hour after application. Use blue^m oral gel after brushing and / or rinsing with blue^m toothpaste / mouthwash or as indicated by your dental professional.

Ingredients:

Aqua, Alcohol, Glycerin, Silica, Sodium Saccharin, Sodium Perborate, Citric Acid, PEG-32, Sodium Gluconate, Cellulose Gum, Xanthan Gum, Lactoferrin.

More information under www.bluemcare.com

In case of questions on the use of blue^m oral gel ask your dental professional. Keep these Instructions for use for the full period of use of blue^m oral gel.

Report to the manufacturer and your local health authority any serious incident in relation to use of this device that may directly or indirectly lead to risk of death or deterioration of health.

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